



NC LOACH

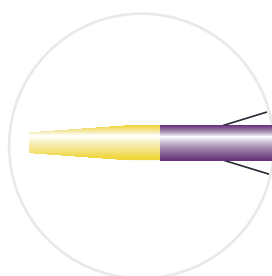


Coronary Balloon Dilatation Catheter

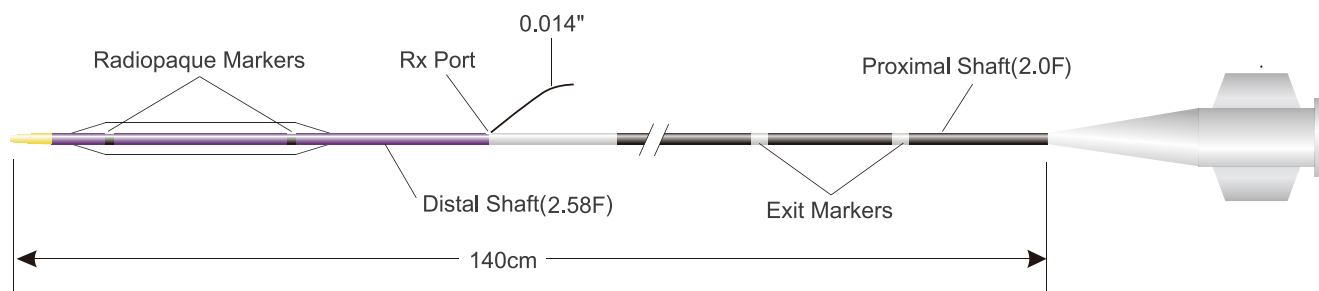

Soft and Rounding TIP

- Enhance the crossability
- Minimize the damage to the vessel wall

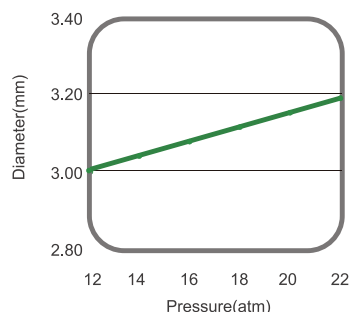


Excellent Balloon Performane

- Controlled diameter growth per atm
- Stabilized properties and RBP value

Tight Memory-Three-Fold Balloon

- Good performance of repeated dilatation
- Slender profile for enhanced crossability

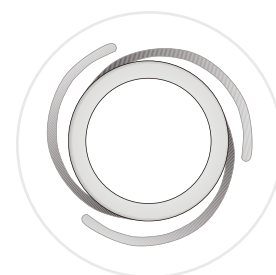

COMPLIANCE CHART:

Diameter(mm) Pressure(atm)/(KPa)	2.00	2.25	2.50	2.75	3.00	3.25	3.50	3.75	4.00	4.50	5.00
6/608	1.94	2.18	2.42	2.65	2.89	3.14	3.38	3.61	3.85	4.29	4.76
8/810	1.96	2.20	2.44	2.69	2.93	3.17	3.42	3.66	3.90	4.36	4.84
10/1013	1.98	2.23	2.47	2.72	2.96	3.21	3.46	3.70	3.95	4.43	4.92
12/1216	2.00	2.25	2.50	2.75	3.00	3.25	3.50	3.75	4.00	4.50	5.00
14/1418	2.02	2.27	2.53	2.78	3.04	3.29	3.54	3.80	4.05	4.57	5.08
16/1621	2.04	2.30	2.56	2.81	3.07	3.33	3.58	3.84	4.10	4.64	5.16
18/1823	2.06	2.32	2.58	2.85	3.11	3.36	3.62	3.89	4.15	4.71	5.24
20/2026	2.08	2.35	2.61	2.88	3.14	3.40	3.66	3.93	4.20	4.77	5.31
22/2229	2.10	2.37	2.64	2.91	3.18	3.44	3.70	3.98	4.25	4.84	5.39
24/2431	2.12	2.39	2.67	2.94	3.22	3.48	3.74	4.03	4.30	4.91	5.47
26/2634	2.14	2.42	2.70	2.97	3.25	3.52	3.78	4.07	4.35	4.98	5.55

Do not exceed the rated burst pressure(RBP)

*

NP-Nominal Pressure

RBP-Rated Burst Pressure